

INSTALL GUIDE

NEW CareLink™
Personal uploader

Upload to CareLink
to more easily partner
with your doctor.

IMPROVE YOUR DOCTOR VISITS BY UPLOADING TO CARELINK



NEW simplified steps:

- Four-step process to add
- Two-step upload




Understand your
glucose patterns




Collaborate with
your doctor


GET STARTED BY DOWNLOADING THE NEW UPLOADER INSTALL FILES

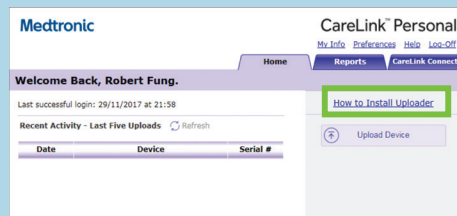

- 1
 - Log In
 - Click "How to Install"

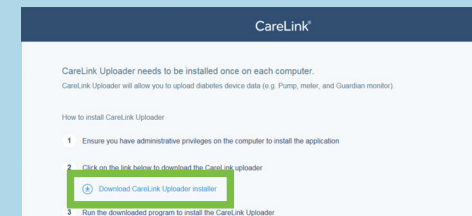


- 2 Click the download link

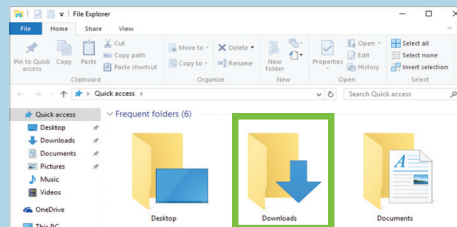


- 3 Find the downloaded file

Medtronic


AFTER DOWNLOADING, INSTALL THE UPLOADER

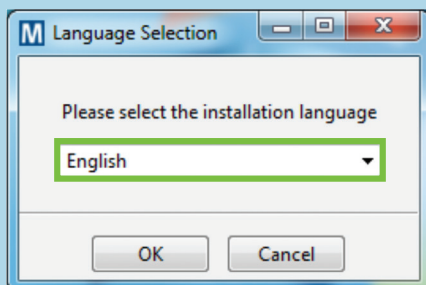


1 Choose your language

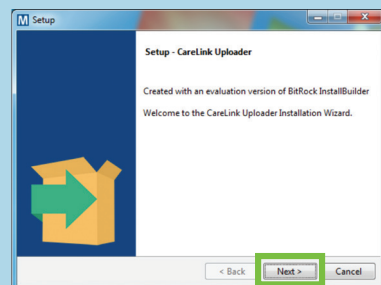

2 Click "Next"

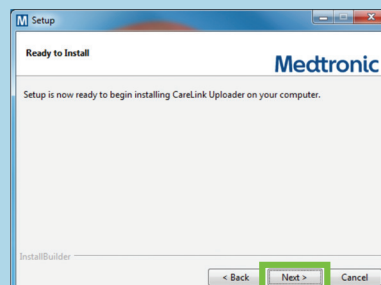


3 Click "Next"

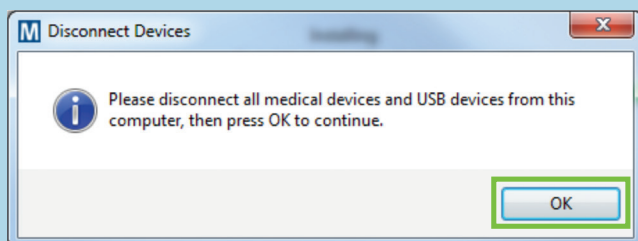


- 4**
- Make sure your CareLink USB or meter are not plugged in and click "OK"
 - Click "Finish" to complete the installation


NOW, YOU'RE READY TO UPLOAD

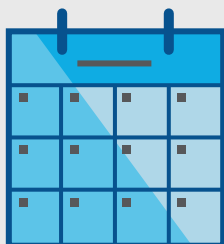

1 Upload regularly

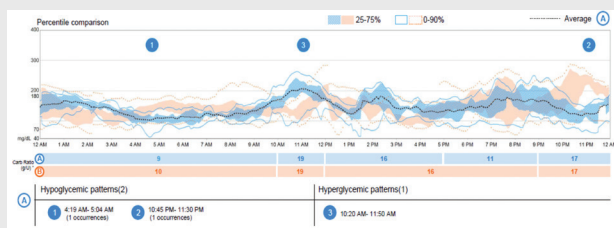

- 2**
- Evaluate improvements
 - Review trends
 - Check Time in Range
 - Identify questions

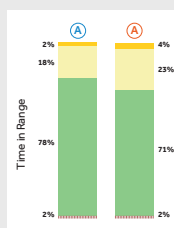

3 Link with your doctor at your next appointment

INSTALL GUIDE

NEW CareLink™ Personal uploader

Safety Information: CareLink™ software

CareLink software is intended for use as an adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself. For more details, please consult the User Guide at medtronicdiabetes.com/support/download-library/user-guides.

© 2018 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. 940M18156-011 19MAR2018

Medtronic Diabetes
18000 Devonshire Street
Northridge, CA 91325
800.646.4633