

UPLOAD GUIDE

CareLink™
Personal uploader

Your first step to
understanding more
about your diabetes.



TO ENSURE SUCCESS, GET EVERYTHING READY

Plug your USB or meter into your computer

- CareLink USB or meter plugged in and placed as close as possible to the pump
- Keep the meter linked with the pump



Make sure your pump is ready to upload

Check that the pump is not on one of these screens.



- No temp basals
- No alarms
- No active bolus
- No low battery

Medtronic

INITIAL FOUR-STEP PROCESS TO ADD A NEW PUMP OR METER



1 Choose pump or meter

Pump




2 Select the pump model

Meter




2 Select the meter brand if you want to upload BG meter only data

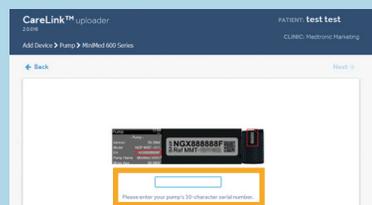

3 Enter pump serial number




3 Select meter model

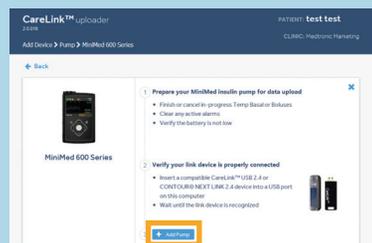

4

- Make sure the pump is ready
- Click "Add Pump"

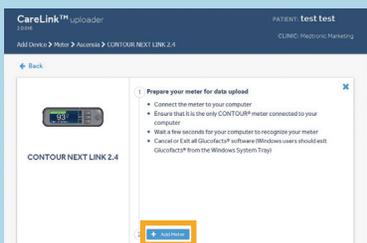


4

- Make sure the meter is ready
- Click "Add Meter"


TWO EASY STEPS TO UPLOAD PUMP OR METER DATA

STEP 1

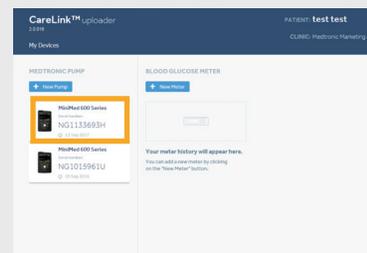

Select the device

STEP 2

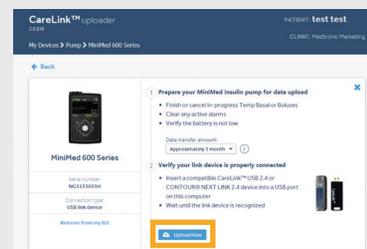

- Make sure the device is ready to upload
- Click "Upload Now"



Link to your doctor's CareLink system for more efficient data sharing.

UPLOAD GUIDE

CareLink™
Personal uploader

Safety Information: CareLink™ software

CareLink software is intended for use as an adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself. For more details, please consult the User Guide at medtronicdiabetes.com/support/download-library/user-guides.

© 2018 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. 940M18157-011 18Mar2018

Medtronic Diabetes
18000 Devonshire Street
Northridge, CA 91325
800.646.4633