Medtronic

Medtronic 18000 Devonshire St. Northridge, CA 91325 www.medtronicdiabetes.com

URGENT MEDICAL DEVICE CORRECTION

MiniMed[™] 600 Series Pump System Communication Issue

Insulin Pump	Model Number
MiniMed™ 630G	MMT-1715, MMT-1755, MMT-1754
MiniMed™ 670G	MMT-1780, MMT-1781, MMT-1782, MMT-1760, MMT-1761, MMT-1762, MMT-1740,
	MMT-1741, MMT-1742

September 2022

Dear Valued Customer:

You are receiving this letter because our records indicate that you may be using a MiniMed[™] 600 series insulin pump. For your safety, we want to inform you of a potential issue associated with the communication protocol used by your pump system. Unauthorized access to your pump's communication protocol could compromise your pump's delivery of insulin. This letter provides actions and mitigations you should take so please carefully review the information below.

ISSUE DESCRIPTION



MiniMed™ 600 series insulin pump¹



Guardian™ Link 3 transmitter



CareLink-

Contour[®] Next Link 2.4 Blood Glucose Meter

4	CareLink™ USB

The MiniMed[™] 600 series pump system consists of components such as the pump, continuous glucose monitoring (CGM) transmitter, blood glucose meter and CareLink[™] USB

¹ Sample image. Pump screen and buttons vary by model.

US-IPT-2200302 © 2022 Medtronic. All rights reserved. Medtronic, and Medtronic logo are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

device that communicate wirelessly. Medtronic has recently identified a potential issue through internal testing whereby, under specific circumstances, the communication between the components of the pump system could be compromised through unauthorized access.

For unauthorized access to occur, a nearby person other than you or your care partner would need to gain access to your pump at the same time that the pump is being paired with other system components. **This cannot be done over the internet.**

Medtronic has no evidence to date that such an issue has occurred. However, in the unlikely event that unauthorized access would be successful, the access could be used to deliver too much or too little insulin through delivery of an unintended insulin bolus or because insulin delivery is slowed or stopped. Too much insulin could result in hypoglycemia (low blood sugar) which can potentially lead to seizure, coma or death. Too little insulin could result in hyperglycemia (high blood sugar) which can potentially lead to diabetic ketoacidosis.

Medtronic recommends all patients take the actions and precautions listed below.

ACTIONS REQUIRED:

 Turn off the "Remote Bolus" feature on your pump if it is turned on. Note that the "Remote Bolus" capability is on by default, so you should take this action even if you have never used this feature. See Appendix: "How to Turn Off Remote Bolus Settings" or by visiting our website at www.medtronicdiabetes.com/remote-bolus.



Please acknowledge that you have read and understood this notification and have

followed the actions and precautions listed in this letter by completing and returning the attached confirmation form by mail.

RECOMMENDED PRECAUTIONS:

- 1. Keep your pump and connected system components within your control at all times.
- 2. Be attentive to pump notifications, alarms, and alerts.
- Immediately cancel any boluses you or your care partner did not initiate, monitor blood glucose levels closely and reach out to Medtronic 24-Hour Technical Support to report the bolus. NOTE: Turning off remote bolus feature will ensure no remote bolus is possible.

US-IPT-2200302 © 2022 Medtronic. All rights reserved. Medtronic, and Medtronic logo are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

- 4. Disconnect the USB device from your computer when you're not using it to download pump data.
- 5. DO NOT confirm remote connection requests or any other remote action on the pump screen unless it is initiated by you or your care partner.



- 6. DO NOT share your pump's or devices' serial numbers with anyone other than your healthcare provider, distributors, and Medtronic.
- 7. DO NOT accept, calibrate, or bolus using a blood glucose reading you didn't initiate.
- 8. DO NOT connect to or allow any third-party devices to be connected to your pump
- 9. DO NOT use any software which has not been authorized by Medtronic as being safe for use with your pump.
- 10. Get medical help immediately when experiencing symptoms of severe hypoglycemia or diabetic ketoacidosis.
- 11. Reach out to Medtronic 24-Hour Technical Support if you suspect a pump setting or insulin delivery have changed unexpectedly, without your knowledge.

As always, we're here to support you. Please contact Medtronic 24-Hour Technical Support at 1-800-646-4633, option 1 if you suspect unauthorized access has occurred or if you experience any adverse events or quality problems with your device.

Adverse reactions or quality problems experienced with the use of this product may also be reported to the FDA's MedWatch Adverse Event Reporting program:

- Submit online: <u>www.fda.gov/medwatch/report.htm</u>
- Submit by regular mail or fax: Download form www.fda.gov/MedWatch/getforms.htm or call 1-800-332-1088 to request a form, then complete and return to the address on the pre-addressed form, or submit by fax to 1-800-FDA-0178.

We understand this may impact your experience and are here to support you. Additional information can be found at <u>www.medtronic.com/security.</u>

The best step you can take now to eliminate your individual risk of unintended delivery of insulin is to permanently turn off the Remote Bolus feature on your pump. We will

US-IPT-2200302 © 2022 Medtronic. All rights reserved. Medtronic, and Medtronic logo are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

continue to actively monitor the situation and are committed to sharing relevant information or actions with you in the future.

Sincerely,

Mona

Chirag Tilara Vice President, Quality Medtronic Diabetes

Robert a. Vigersky, M.D.

Robert Vigersky, M.D. Chief Medical Officer Medtronic Diabetes

APPENDIX

How to Turn Off Remote Bolus Settings

Follow these steps to turn off Remote Bolus feature:

<u>MiniMed[™] 670G Insulin Pump (MMT-1780, MMT-1781, MMT-1782)</u>

Note: The Remote Bolus feature is not available when in SmartGuard™ Auto Mode

- 1. If Auto Mode is active, Auto mode must first be deactivated to enter manual mode using the following step:
 - Press Center Button (🔘) and go to the Auto Mode settings screen.

Options>SmartGuard>Auto Mode

Enter BG	0	Options	SmartGuard	Auto Mode	Auto Mode
Temp Target	28	SmartGuard	Auto Mode	Auto Mode 🚨	Auto Mode
Audio Options	8	History	High Setup	Auto Mode BG alert	Auto Mode BG alert 🚢
Status		Reservoir & Tubing	Low Setup		
Suspend Delivery	0	Delivery Settings	Snooze		
Options	۵.	Event Markers		Save	Save

- 2. Select Save
- 3. In manual mode, Press Center Button (\bigcirc) and go to the Remote Bolus screen.

Options>Utilities>Remote Bolus

Basal	_ بي	Options	Utilities
Audio Options	8.	Reservoir & Tubing	Sensor Settings
Status	Ë	Delivery Settings	Airplane Mode
Suspend Delivery		Event Markers	Display Options
Options	(Reminders	Time & Date
Training Demo		Utilities	Remote Bolus

• The Remote Bolus screen appears



4. Select **Remote Bolus** to turn the feature off.

US-IPT-2200302 © 2022 Medtronic. All rights reserved. Medtronic, and Medtronic logo are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.



5. Select Save

• If you wish to reactivate Auto Mode, go to the Auto Mode settings screen.

Options>SmartGuard>Auto Mode

Enter BG	0	Options	SmartGuard	Auto Mode
Temp Target	- 28	SmartGuard	Auto Mode	Auto Mode 🛛 🕰
Audio Options	8	History	High Setup	Auto Mode BG alert 斗
Status	Ë	Reservoir & Tubing	Low Setup	
Suspend Delivery		Delivery Settings	Snooze	
Options		Event Markers		Save

MiniMed[™] 630G (MMT-1714, MMT-1715)

Note: Remote Bolus default setting is "ON".

1. Press Menu Button (1) and go to the Remote Bolus screen.

Menu>Utilities>Remote Bolus

Options	Utilities
Reservoir & Tubing	Sensor Settings
Delivery Settings	Airplane Mode
Event Markers	Display Options
Reminders	Time & Date
Utilities	Remote Bolus

a. The Remote Bolus screen appears.



- 2. Select Save
- 3. Select Remote Bolus to turn the feature off



4. Select Save

Medtronic

Medtronic 18000 Devonshire St. Northridge, CA 91325 www.medtronicdiabetes.com

Customer Confirmation Form

URGENT MEDICAL DEVICE CORRECTION

MiniMed[™] 600 Series Pump System Communication Issue

Insulin Pump	Model Number
MiniMed™ 630G	MMT-1715, MMT-1755, MMT-1754
MiniMed™ 670G	MMT-1780, MMT-1781, MMT-1782, MMT-1760, MMT-1761, MMT-1762, MMT-1740,
	MMT-1741, MMT-1742

By signing this form, I confirm that I have read and acknowledged the Urgent Medical Device Correction letter dated September 2022 from Medtronic regarding MiniMed[™] 600 Series Pump System Communication Issue and have taken all the appropriate actions and precautions listed in the letter.

Please complete and sign the form as indicated below and return by mail using the attached prepaid envelope. You can also email this form to our Field Corrective Action Department at **rs.safetyinformation@medtronic.com** or send via fax to **1-888-844-8084** to the attention of **FCA Department. After we receive your response, we will not send additional copies of this initial notice.**

Name	First Name						
		La	st Name				
Select one	🗆 Pump user	🗌 Care part	ner	□ Other: _			
Signature			Date	/ [/		YYYY	
Telephone			<u> </u>				
Notes							

For questions, contact our 24-Hour Technical Support line at 1-800-646-4633, option 1.

As always, thanks for your support.