

MEDICAL EMERGENCY CARD



I wear a required medical device prescribed by my physician.

Medtronic

In an emergency, please contact

Physician name:

Phone number:

Emergency contact:

Phone number:

I have diabetes

If my behavior is peculiar, if I appear intoxicated, or if I am unconscious, it may be a result of severe low blood sugar.

I am not intoxicated Call for medical assistance

If I am awake and able to swallow, give me a source of sugar (for example: juice, candy, or non-diet soft drink). Do not try to give me food or drink if I am unconscious.

Healthcare professionals

I wear an insulin pump.

- The pump delivers rapid-acting insulin at a constant rate.
- To stop insulin delivery on the pump:
 - Press any button to wake the pump.
 - Press Ⓞ.
 - Press the highlighted button shown on the screen to unlock the pump.
 - Press ∨ to highlight **Suspend Delivery**.
 - Press Ⓞ.
 - Press > to highlight **Yes**.
 - Press Ⓞ to suspend insulin delivery.
- **Do not** remove the pump battery.
- **Do not** remove the pump without medical consent.
- If the pump sounds an alarm, follow the instructions on the pump screen or call the 24-Hour Technical Support at 1-800-646-4633.

Airport information



I wear a required medical device prescribed by my physician.

Medtronic

Airport security

- Because travel rules are subject to change, it is advisable to check with the Transportation Safety Administration (TSA) before traveling. TSA information is found at <http://www.tsa.gov/travel/specialprocedures> or by calling 1-866-289-9673.
- The pump **must not** go through the x-ray machine that is used for carry-on or checked luggage.
- The full body scanner is also a form of x-ray. If you choose to go through the full body scanner, you must remove your insulin pump and disconnect the infusion set at the insertion site. If you use continuous glucose monitoring (CGM), you must also remove your sensor and transmitter before the scan.
- To avoid removing your devices, you must request an alternative screening process that does not use x-ray.
- The insulin pump, infusion set, reservoir, and CGM system can withstand exposure to airport metal detectors used at airport security checkpoints.

In flight

The MiniMed™ 770G insulin pump and system devices are suitable for use in aircraft. When flying in an aircraft, it is important that you keep your pump connected to your body and check your blood glucose levels frequently.

Patient information

My name:

My address:

Phone number:

Medical device information

Medical device type:

Device serial number:

© 2019 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

Medtronic



Medtronic MiniMed
18000 Devonshire Street
Northridge, CA 91325 USA
1800 646 4633
+1818 576 5555
www.medtronicdiabetes.com

M998594A011_A