

Medtronic

InPen™ Insights Reports for healthcare providers


To enable your healthcare professional to see your InPen Insights Reports in their clinic CareLink™ software, there are two actions for you to complete:

1. Link your CareLink Personal account with your InPen app
2. Link your CareLink Personal account with their CareLink software in the clinic



If you do not have a CareLink Personal account, you can sign up at carelink.minimed.com You can also reset your password if you forgot it.

Please turn over to see how you can link your accounts.

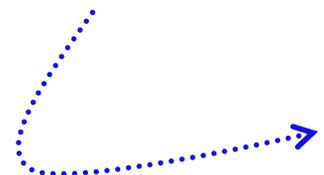


1

How to link your InPen app with your CareLink Personal account

If your InPen app is not yet linked with your CareLink Personal account please make sure your app is the latest version and follow the below steps in the InPen app.

Note: This link will take you to sign up for an account if you have not already

Representative profile

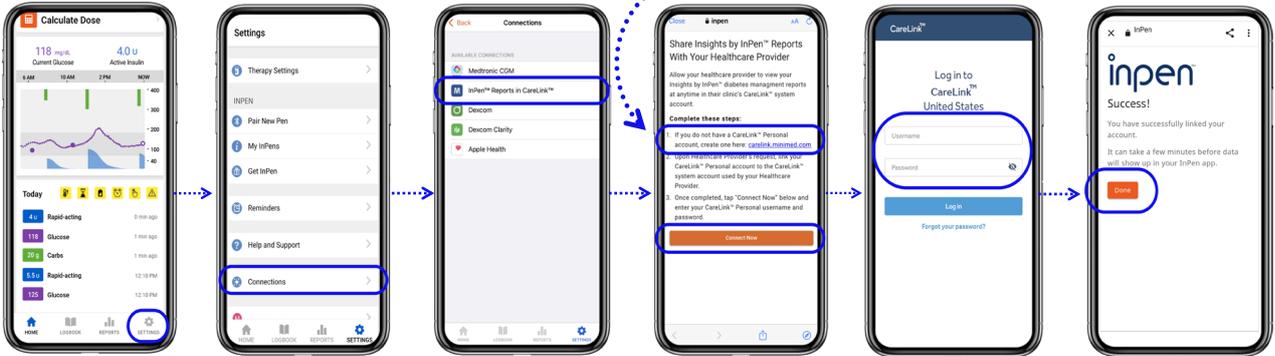

1. Enter the app and tap on **Settings**

2. Tap on **Connections**

3. Tap on **InPen Reports in CareLink**

4. Tap **Connect Now**, and read the terms and conditions then tap **Continue**

5. Enter your CareLink Personal username and password



6. Tap **Done** to finish the process and return to the home screen

You can stop sharing at any point from the CareLink Personal Data Sharing tab*

2

How to link your CareLink Personal account with your HCP

- Provide your CareLink Personal username (password not required) to your HCP.
- Your HCP can then request linking the accounts but you have to agree in CareLink Personal.
- Once the link has been requested by your HCP you will receive an email that will ask you to go to your CareLink Personal account to approve the link.

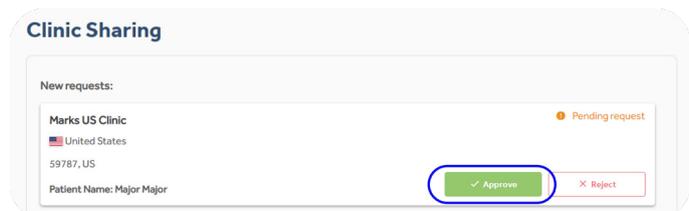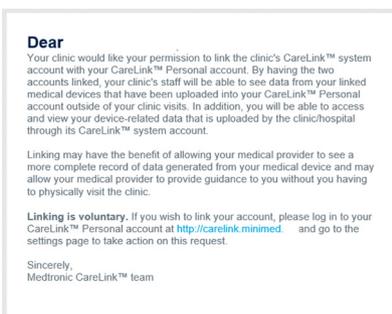

*Your HCP will retain all previously shared CareLink™ data, InPen™ data is not stored in CareLink™ software. All HCP access to the InPen™ data is ended when you stop sharing. InPen requires a prescription and proper settings, or you could experience highs/lows. For more, see bit.ly/InPenSafety

© 2023 Medtronic. All rights reserved. Medtronic, Medtronic logo and Engineering the extraordinary are trademarks of Medtronic. Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

US-SOF-2300004