

# Wallet Card

**Medtronic**  
MiniMed™ 780G




**Medtronic**

**Medtronic MiniMed**  
18000 Devonshire Street  
Northridge, CA 91325 USA  
1800 646 4633  
+1818 576 5555  
[www.medtronicdiabetes.com](http://www.medtronicdiabetes.com)


M058215C001\_2

## Main Menu

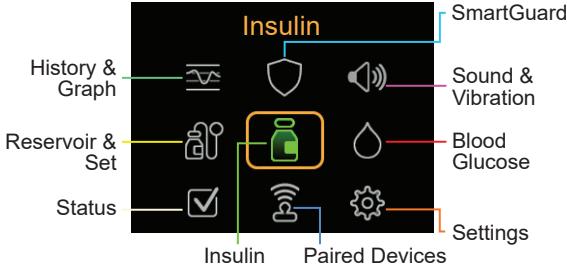



<p><b>History &amp; Graph</b></p> <p>History Sensor Glucose Review* Graph Time in Range*</p>	<p><b>SmartGuard</b></p> <p>SmartGuard Checklist Temp Target* SmartGuard Settings* SmartGuard*</p>
<p><b>Sound &amp; Vibration</b></p> <p>Silence Sensor Alerts* Volume Sound Vibration Alert Settings</p>	<p><b>Reservoir &amp; Set</b></p> <p>New Reservoir &amp; Set New Reservoir Only New Set Only Fill Cannula*</p>
<p><b>Insulin</b></p> <p>Bolus Basal Suspend All Delivery Delivery Settings</p>	<p><b>Blood Glucose</b></p> <p>Enter BG</p>
<p><b>Status</b></p> <p>Suspend All Delivery SmartGuard Checklist Pump Sensor*</p>	<p><b>Paired Devices</b></p> <p>Pair New Device Pair CareLink Mobile** Meter** CGM**</p>
<p><b>Settings</b></p> <p>Alert Settings Delivery Settings Device Settings</p>	<p><i>*Menu choices appear grey when certain settings are turned off or are otherwise not available.</i> <i>**Menu choices appear when the device is paired to the pump.</i></p>

## Home screen in SmartGuard™ feature

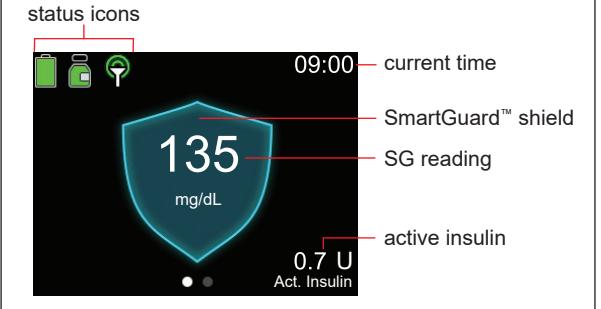

## Stopping and resuming insulin delivery

<p><b>Stopping all insulin delivery</b></p> <p>The Suspend All Delivery option stops all insulin delivery (basal and bolus).</p>	<p><b>To stop all insulin delivery:</b></p> <ol style="list-style-type: none"> <li>From the Home screen, press  and select .</li> <li>Select <b>Suspend All Delivery</b>.</li> <li>Select <b>Yes</b> when prompted.</li> </ol>
<p><b>Resuming basal insulin delivery</b></p> <p>The Resume Basal option restarts the basal delivery after the pump has been suspended. The Resume Basal option does not restart a bolus delivery.</p>	<p><b>To resume basal insulin delivery:</b></p> <ol style="list-style-type: none"> <li>From the Home screen, press  and select .</li> <li>Select <b>Resume Basal</b>.</li> <li>Select <b>Yes</b> when prompted.</li> </ol> <p><b>NOTE:</b> Basal insulin delivery can be manually resumed during a Suspend before low or Suspend on low event.</p>
<p><b>Stopping a bolus delivery</b></p> <p>The Stop Bolus option stops a bolus delivery only. The Stop Bolus option does not stop the basal delivery.</p>	<p><b>To stop a normal bolus, or the Now portion of a Dual Wave™ bolus:</b></p> <ol style="list-style-type: none"> <li>From the Home screen, press  and select .</li> <li>Select <b>Stop Bolus</b>.</li> <li>Select <b>Yes</b> when prompted.</li> <li>Select <b>Done</b>.</li> </ol> <p><b>To stop a Square Wave™ bolus or the Square portion of a Dual Wave™ bolus:</b></p> <ol style="list-style-type: none"> <li>From the Home screen, press  and select .</li> <li>Select <b>Bolus</b>.</li> <li>Select <b>Stop Bolus</b>.</li> <li>Select <b>Yes</b> when prompted.</li> <li>Select <b>Done</b>.</li> </ol>
<p><b>Manually resuming basal insulin delivery during a Suspend before low or Suspend on Low event.</b></p> <p>If unable to wait for the pump to automatically resume the basal insulin during a Suspend before low or Suspend on low event, manually resume the basal delivery.</p>	<p><b>To manually resume basal insulin delivery:</b></p> <ol style="list-style-type: none"> <li>From the Home screen, press  and select .</li> <li>Select <b>Resume Basal</b>.</li> <li>Select <b>Resume Basal</b> again.</li> <li>Select <b>Yes</b> when prompted.</li> </ol>

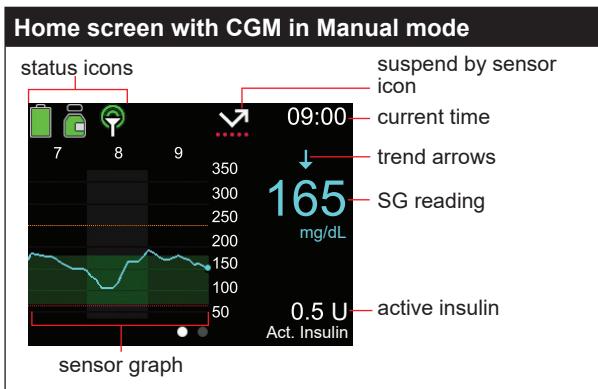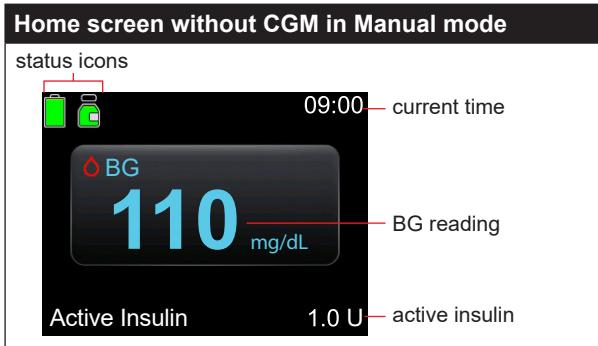


© 2024 Medtronic. Medtronic and Medtronic logo are trademarks of Medtronic.  
™\* Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

MiniMed™, SmartGuard™, Dual Wave™, and Square Wave™ are trademarks of Medtronic MiniMed, Inc.

# Wallet Card

Medtronic  
MiniMed™ 780G

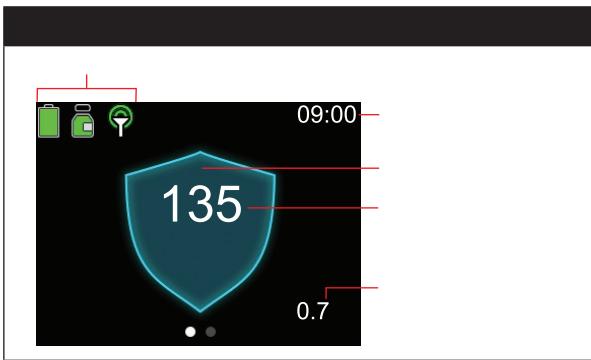


© 2024

