**Español**

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**English**

**Indications for Use**  
This reservoir is indicated for the subcutaneous infusion of medication, including insulin, from compatible Medtronic insulin pumps and infusion sets. Refer to your Medtronic insulin pump user guide for compatibility.

**Contraindications**  
This reservoir is contraindicated for the infusion of blood or blood products.

**Warnings**  
Reservoir and transfer guard are sterile, nonpyrogenic, and for single use only. Do not clean or resterilize. Reuse of the reservoir may lead to any one of the following: insulin degradation, infection, inaccurate medication delivery, or leaks which may cause damage to the pump. Do not use if package has been opened or damaged. Dispose of transfer guard safely in sharps container. Failure to remove trapped air from reservoir may result in inaccurate delivery of medication.

**Indications for Use**  
If insulin, or any liquid, gets inside the tubing connector, it can temporarily block the vents that allow the pump to properly fill the infusion set. This may result in the delivery of too little or too much insulin, which can cause hypoglycemia or hyperglycemia. If this occurs, start over with a new reservoir and infusion set.

**Precautions**  
Insulin should be at room temperature before filling reservoir. Once reservoir is filled, use promptly. Do not check for leaks after changing the reservoir and infusion set. Conduct your first setup in the presence of your healthcare professional.

**Warranty**  
Visit <http://www.medtronicdiabetes.com/warranty>.  
©2015 Medtronic MiniMed, Inc. All rights reserved.  
MiniMed® is a registered trademark of Medtronic MiniMed, Inc.

# MiniMed® Reservoir

Reservoir • Reservorio

# MiniMed® Reservoir

**Medtronic**

**Medtronic MiniMed**  
18000 Devonshire Street  
Northridge, CA 91325  
USA  
800 646 4633  
818 576 5555  
[www.medtronicdiabetes.com](http://www.medtronicdiabetes.com)

Rx Only

D9195886-2AF2\_a

(01)00643169591714(10)NA

**EN** Refer to pump and infusion set user guides for installation and filling.

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