ONE-PRESS SERTER


User Guide
Guía del usuario
用户指南

Medtronic

Introduction

The one-press serter (MMT-7512) is used as an aid to insert Medtronic glucose sensors.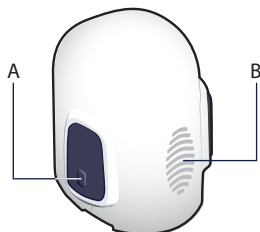

- A. Bump (on both buttons)
- B. Thumbprint marking

Indications for use

The serter is indicated for single-patient use. It is not intended for use by multiple patients. Refer to the system user guide for additional information, such as the intended age group.

Contraindications

No contraindications are associated with serter use. For contraindications related to continuous glucose monitoring, see the system user guide.

Clinical benefits

Not applicable. The serter is an accessory that helps to insert compatible Medtronic sensors.

User Safety

Warnings

Read this entire user guide prior to sensor insertion. The serter does not work the same way as other Medtronic insertion devices. Failure to follow directions may result in improper insertion, pain, or injury.

For additional warnings related to sensor use, see the sensor user guide.

Keep the serter out of reach of children. The serter contains small parts and may pose a choking hazard that can result in serious injury or death.

Healthcare professionals and caregivers:

- A retractable needle is attached to the sensor. Minimal bleeding may occur. Always wear gloves to insert the sensor
- Cover the sensor with sterile gauze to remove the needle housing from the sensor

Place the needle housing directly into a sharps container after sensor insertion to prevent accidental needle stick injury.

Always inspect the sensor packaging for damage before use. Sensors are sterile and non-pyrogenic unless the package has been opened or damaged. If the sensor packaging is open or damaged, discard the sensor directly into a medical waste container. Use of a non-sterile sensor can result in infection at the insertion site.

For questions or concerns related to serter use, contact 24-Hour Technical Support for assistance.
For medical questions or concerns, contact a healthcare provider.

Assistance

Department	Telephone Number
24-Hour Technical Support (calls within the United States)	+1 800 646 4633
24-Hour Technical Support (calls outside the United States)	+ 1 818 576 5555
Website	www.medtronicdiabetes.com

Precautions

Wash hands with soap and water prior to sensor insertion to reduce the risk of infection.

Inspect the insertion site for bleeding under, around, or on top of the sensor. If bleeding occurs, use a sterile gauze pad to apply steady pressure to the sensor for three minutes.

WARNING: Do not use non-sterile gauze to apply pressure to the sensor. Use of non-sterile gauze can cause infection at the insertion site.

If bleeding stops after three minutes of steady pressure, connect the transmitter to the sensor.

CAUTION: Do not connect the transmitter to the sensor if bleeding at the insertion site does not stop. Active bleeding can damage the transmitter and result in a loss of communication between the transmitter and the receiving device.

If bleeding does not stop after three minutes of steady pressure, remove the sensor, and continue to apply pressure until bleeding stops. Discard the used sensor directly in a sharps container after removal. Insert a new sensor at a different insertion site.

Note: Always use a new insertion site to insert a new sensor. Reuse of the same insertion site can cause redness, irritation, pain, bleeding, tenderness, and inflammation.

Do not clean, resterilize, or attempt to remove the needle from the needle housing. Needle stick injury may occur.

Do not reuse the sensor. Reuse of the sensor can cause damage to the sensor surface and lead to inaccurate sensor glucose values, irritation, and infection at the insertion site.

Rotate the sensor insertion site so that sites do not become overused.

Confirm that the sensor is firmly attached to the skin to prevent it from coming out of the insertion site. Factors that increase the likelihood of the sensor coming out include increased physical activity (particularly in younger patients) and improper taping technique.

Risks and side effects

The serter contains small parts and may pose a choking hazard that can result in serious injury or death.

Side effects include discomfort and skin irritation at the insertion site.

Hazardous substances

None.

Allergens

None known.

Where to insert the sensor

Choose an insertion site with an adequate amount of subcutaneous fat.

For optimal sensor performance, and to prevent accidental sensor removal:

- Do not insert the sensor into muscle, tough skin, or scar tissue
- Avoid areas that are constrained by clothing or accessories
- Avoid areas subjected to vigorous movement during exercise

For a diagram of recommended sensor insertion areas, see the sensor user guide.

Inserting the sensor

For instructions on how to insert the sensor, see the sensor user guide.

Maintenance

Cleaning

Clean the serter after every use or whenever debris, blood, or other contaminants are visible.

<p>To clean the serter:</p> <p>Note: Press and release the mechanism on the underside of the serter to fully rinse the device.</p>
1. Rinse the serter under running tap water.
2. Submerge the serter in a solution of mild soap and warm water.
3. Use a soft-bristled brush to clean the serter until it is free of visible dirt.
4. Rinse the serter under running tap water until all visible soap is gone. Make sure to fully rinse all hard-to-reach areas.
5. Shake off any excess water. Place the serter upright on a clean cloth to air dry.

Storage

Store the serter in the released position at room temperature.

Disposal

Dispose of the serter according to local regulations.

Technical specifications

Approximate Dimensions	Approximate Weight
3.09 x 2.72 x 2.27 inches (7.85 x 6.91 x 5.77 centimeters)	3.97 ounces (112.5 grams)

Icon table

	Consult instructions for use
	Caution
	Batch code
	Catalogue number
	One per container/package
	Storage humidity limit
	Storage temperature limit
	Manufacturer
	Date of manufacture
	Requires prescription in the USA

©2023 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademark of a Medtronic company.

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