

Medical Emergency Card



Medtronic

© 2011 Pearson Education, Inc. or its affiliate(s). All rights reserved. No part of this publication may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or by any information storage or retrieval system, without prior written permission from the publisher.



I wear a required medical device prescribed by my physician.

Medtronic

In an emergency, please contact

Physician name:

Phone number:

Emergency contact:

Phone number:

I have diabetes

If my behavior is peculiar, if I appear intoxicated, or if I am unconscious, it may be a result of severe low blood sugar.

**I am not intoxicated
Call for medical assistance**

If I am awake and able to swallow, give me a source of sugar (for example: juice, candy, or non-diet soft drink). Do not try to give me food or drink if I am unconscious.

Healthcare professionals

I wear an insulin pump.

- The pump delivers rapid-acting insulin at a constant rate.
- To stop insulin delivery on the pump:
 - Press any button to wake the pump.
 - From the home screen, press \wedge to go to the status screen.
 - Press \odot to **Suspend All Delivery**.
 - Press \triangleright to highlight **Yes**.
 - Press \odot to suspend all insulin delivery.
- **Do not** remove the pump battery.
- **Do not** remove the pump without medical consent.
- If the pump sounds an alarm, follow the instructions on the pump screen or contact a local Medtronic support representative for further assistance at 1-800-646-4633.