English

Indications for use

This product is indicated for the subcutaneous infusion of insulin from an infusion pump.

Contraindications

Infusion sets are indicated for subcutaneous use only and for intravenous (IV) infusion or the infusion of blood or blood products.

Warnings

Inadequate medication delivery, infection and/or site irritation may result from improper insertion and maintenance of the infusion site.

Before insertion, clean the insertion site with isopropyl alcohol.

Carefully remove the needle guard before inserting the infusion set. Do not bend the needle prior to insertion, and do not use the infusion set if the needle is bent or has been damaged.

If using this infusion set for the first time, do the first setup in the presence of your healthcare professional.

Do not leave air in the infusion set. Prime completely.

Check frequently to make sure the needle remains firmly in place as you may not feel pain if it pulls out. The needle must always be completely inserted to receive the full amount of medication.

If the infusion site becomes inflamed, replace the set, and use a new site until the first site has healed. Replace the infusion set if the tape becomes loose, or if the needle becomes fully or partially dislodged from the skin.

Replace the infusion set every 48 hours, or per your healthcare professional's instructions.

Never prime the set or attempt to free a clogged line while the set is inserted. You may accidentally inject too much medication.

Do not put disinfectants, perfumes, or deodorants on the infusion set as these may affect the integrity of the set.

Dispose of the infusion set and needle safely, in a sharps container, after a single use. Do not clean or re-sterilize.

This device is sterile and non-pyrogenic unless the package has been opened or damaged.

Do not use if the package has been opened or damaged. (For Paradigm users, do not use the infusion set if the tubing connector needle has

been damaged.)

Store infusion sets in a cool, dry place. Do not leave infusion sets in direct sunlight or inside a vehicle.

Use aseptic techniques when temporarily disconnecting the set and consult your healthcare provider on how to compensate for missed medication when disconnected.

If infusing insulin, carefully monitor your blood glucose levels when disconnected and after reconnecting.

If infusing insulin, and your blood glucose level become unexplainably high, or an occlusion alarm occurs, check the clogs and/or leaks.

If in doubt, change infusion set because there may be a dislodged needle or partially clogged. Should any of these problems arise, make a plan with your healthcare professional for rapidly replacing insulin. Test your blood glucose level to make sure the problem is corrected.

If infusing insulin, do not change the infusion set just before bedtime unless you check your blood glucose 1-3 hours after insertion.

Reuse of the infusion set may cause damage to the cannula/needle and lead to infection, site irritation, and/or inaccurate medication delivery.

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