


To Rewind the Piston in Pump:




1. Wash your hands. Remove old infusion set from your body and the reservoir from pump.
2. Go to **Reservoir + Set** in **MAIN MENU**, then select **Reservoir Setup** to open **REWIND** screen. Press **ACT** to rewind.

To Fill the Reservoir:


1. Remove reservoir from package.

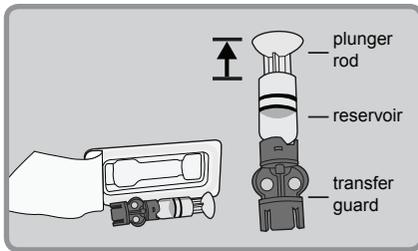


2. Pull plunger rod down to fill reservoir with air.

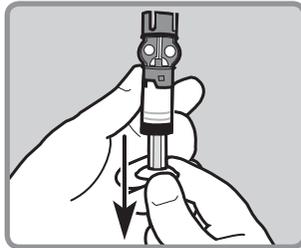


3. After wiping top of insulin vial with alcohol, hold blue transfer guard and press it down onto insulin vial until fully pushed into place.

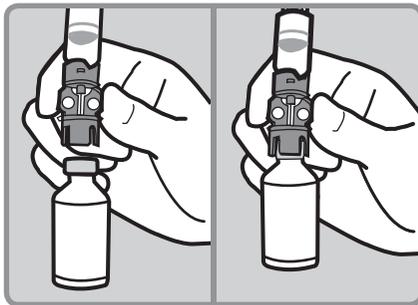

4. Use thumb to push air from reservoir into insulin vial. Continue to hold plunger rod down.

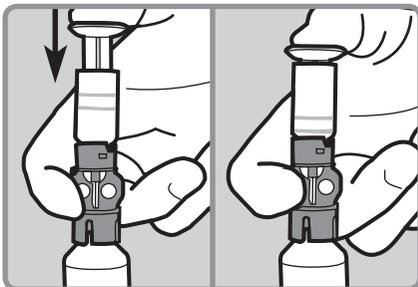


5. Continue to hold plunger rod down as you turn vial over (vial should now be on top and reservoir on bottom). Release plunger. Then slowly pull plunger rod down to fill reservoir with insulin.

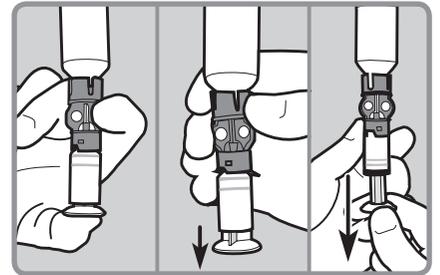


6. Check reservoir for air bubbles. Tap side of reservoir to force bubbles to rise to top of reservoir. Push plunger rod up to move air bubbles into insulin vial.



7. After air bubbles are removed, pull plunger rod back down to fill reservoir with enough insulin to last 2-3 days.


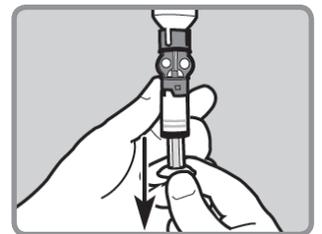


8. Flip vial over so it is now on bottom. Hold blue transfer guard with one hand. With other hand turn reservoir counter-clockwise and pull it straight up until disconnected from blue transfer guard. Disconnect blue transfer guard and dispose into a sharps container.

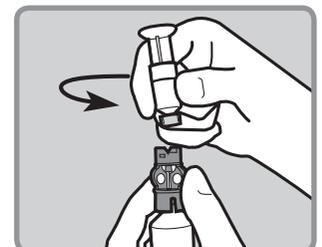


To Connect the Tubing:


1. Remove the mio™ infusion set from packaging by pulling down the red plastic tab. Then peel sterile paper from the bottom.

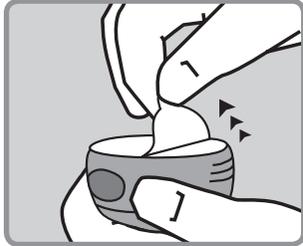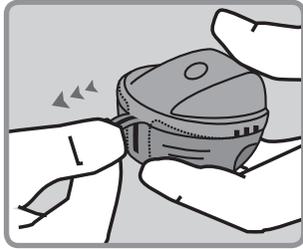


2. With one hand press the three raised markings on the sides of the mio™ lid. Then raise and remove lid.

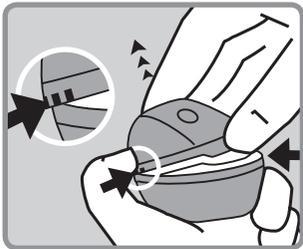

3. Free tubing from slot that holds it in place. Gently unwind tubing in counter clockwise direction.


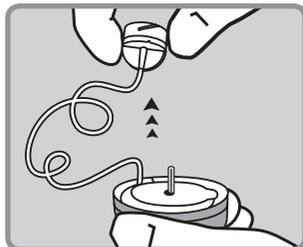


4. Hold reservoir in one hand. With other hand gently push connector onto top of reservoir and turn clockwise until it slides and locks into place.

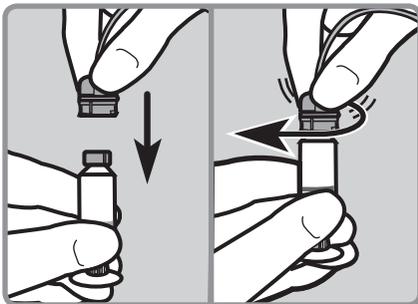


To Connect the Tubing:

5. Tap side of reservoir to force any remaining air bubbles to rise to top. Push up on plunger rod until bubbles are out and small amount of insulin is in tubing.

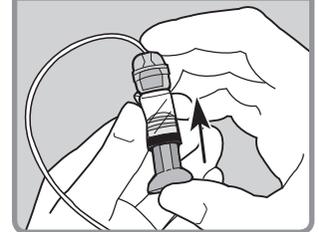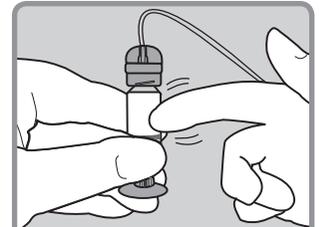


6. Turn plunger rod counter-clockwise until it unscrews from reservoir. Be careful not to pull plunger rod out from reservoir or insulin will spill out.

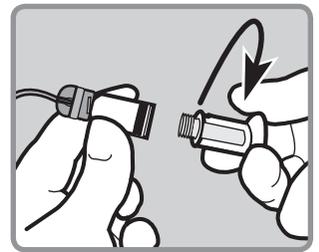


To Fill the Tubing:

1. Press **ACT** so that **REWIND COMPLETE** appears. Insert newly filled reservoir into pump's reservoir compartment.

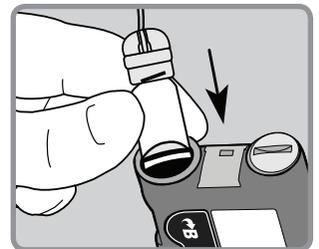


2. Turn reservoir clockwise until it locks in place.

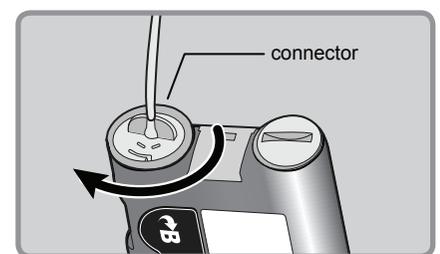


To Fill the Tubing:


- Press **ACT** to go to **FILL TUBING** screen. Be sure infusion set is NOT connected to your body. Then select **Yes** and press **ACT**.

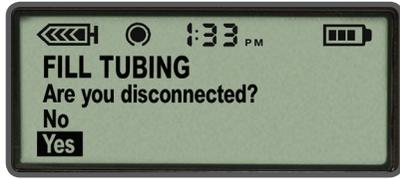


- Press and hold **ACT** to fill tubing. The pump will beep as tubing fills. Release **ACT** when you see drops at tip of connector needle. Then press **ESC**.

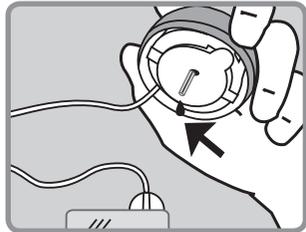


- If you see drops at end of mio™ needle and there are no air bubbles in the tubing, select **Yes** and press **ACT**.

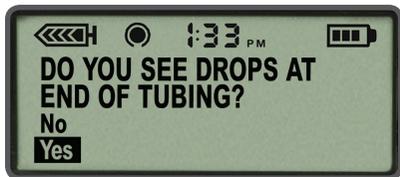

- When the **FILL CANNULA** screen appears, you are ready to insert the mio™ infusion set.

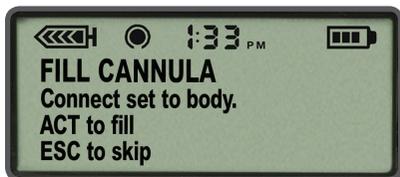

Choose Insertion Site:

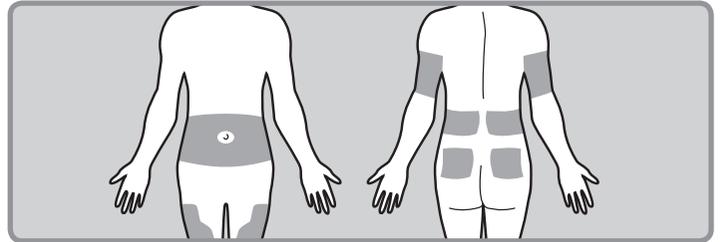

The best areas of the body to insert infusion set are shown in shaded areas of this drawing.


Clean the selected insertion site with alcohol or other antiseptic wipe.

To Insert the mio™ Infusion Set:


- Gently peel paper to expose adhesive.




- Turn mio™ over and hold it by the lined ridges on the sides.

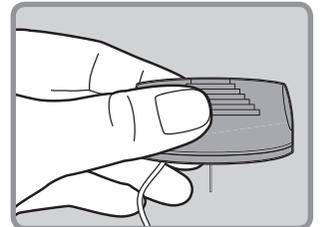

- With other hand pull up on center of inserter until it clicks and locks into place.

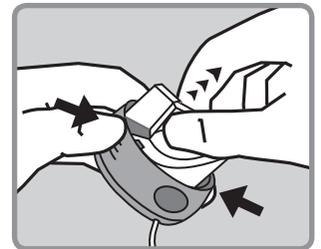


- Carefully hold needle guard by tip. To remove it, gently turn needle guard & pull away to expose needle.

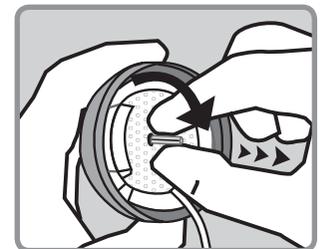


To Insert the mio™ Infusion Set:

- Place tubing in slot on side of inserter.


Caution: Be careful not to pull the tubing tightly when placing it in the slot.

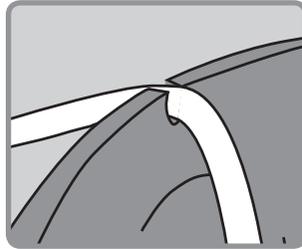


- Place mio™ against prepared site on body. Press the two round indentations on side of mio™ to insert needle.

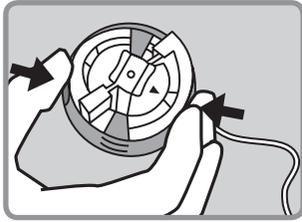

- Push down on center of inserter to press adhesive against skin.

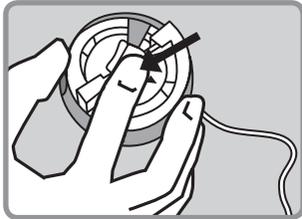

- To remove inserter, hold the center handle and gently pull it straight out away from body. Press adhesive securely against skin.

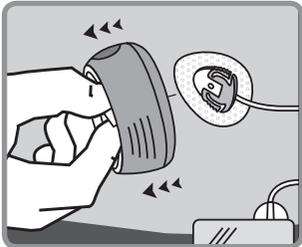


- Place lid back onto inserter to safely discard needle.



To Fill Cannula:

- At **FILL CANNULA** screen, press **ACT**.

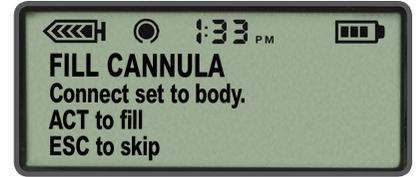

- In order to fill empty cannula under skin press **▲** to enter 0.3 of insulin (if you are using a 6mm cannula) or 0.5 units (for 9mm cannula). Press **ACT**.



OR

